

OSTEOINDUCTAL®

INTENSIVE STIMULATIVE - REGENERATIVE. ORIGINAL!

OBTURAL® Notes on use – please read carefully

PRESENTATION

OBTURAL® is an oleaginous calcium hydroxide suspension in kneadable consistency. The product is filled into a rotating plunger syringe and packed sterilely. One pack contains 1 syringe with 2.5 ml. The medical product is slightly radio-opaque.

INDICATION

OBTURAL® has an intensively antibacterial, inflammation-inhibiting and pain-relieving effect on soft tissue defects. Because of its elevated calcium hydroxide concentration, it may be used exclusively as superficial protective cover for the temporary treatment of soft tissue wounds that cannot be covered plastically or only with difficulty in oral surgery (e.g. mouth-antrum connections or as superficial protective cover after immediate implantations). It acts especially effectively if the product is protected mechanically in addition with the aid of interim prostheses, temporary appliances, Miniplast splints etc.

CONTRAINDICATION

OBTURAL® may never be introduced into deeper and/or closed bone defects. The consequence of such faulty application can be pains and intensive swellings as well as locally limited inhibition of bone regeneration.

INTERACTIONS

The bone wound must not be pretreated with hydrogen peroxide when kneadable OBTURAL® is used. In such a case the bone wound must be rinsed several times with saline solution before application.

SIDE EFFECTS

In rare cases exacerbations and medium-severe oedemas can occur if the product has been applied under pressure. If the soft tissue defects lie in the area of not stabilized gingiva, there can also be oedemas because more intensive extravasation.

STORAGE AND GENERAL NOTE

OBTURAL® should be stored at 4-10 °C, transport is possible at room temperature (20-25 °C). Do not use after the end of the use-by date!



OSTEOINDUCTAL®

INTENSIVE STIMULATIVE - REGENERATIVE. ORIGINAL!

Material that has run out or is stored too warm can become rancid and therefore change its colour and harden.

REUSE

Kneadable OBTURAL® is intended for one-time use. Resterilization is not possible! Any contact with blood, saliva or tissue excludes use in other patients. With strict compliance with the sterility requirements and in the case of need smaller amounts of the kneadable mass can be taken several

times with sterile instruments from the rotating plunger syringe. The responsibility for the reuse of this syringe lies with the dentist.

INSTRUCTIONS FOR PATIENTS

The normal instructions as after every surgical operation.
The patient should avoid especially rinsing the wound or sucking on it.

Status: 04/2006

