ENGLISH INSTRUCTIONS FOR USE

Vial
Remove vial from sterile package using sterile technique. Dispense desired quantity of 4BONE BCH granules into a sterile dish.

Syringe
Remove syringe from sterile package using sterile technique.

Description of 4BONE™ BCH

4BONE BCH is a Resorbable Bone Graft Substitute. 4BONE BCH is a microporous and macroporous biphasic calcium phosphate ceramic consisting of 60% Hydroxyapatite (HA) and 40% beta-Tricalcium Phosphate (β-TCP).

Indications

4BONE BCH is intended for use as a bone grafting material to fill, augment or reconstruct periodontal or oral/maxillofacial defects.

- Typical uses include:
1. Periodontal / Intra-bony defects
2. Extraction sites (implant preparation/ placement)
3. Cystic cavities
4. Ridge augmentation
5. Sinus lifts

Contra-indications

4BONE BCH has limited initial mechanical properties. Therefore, this product is contraindicated where the device is intended as structural support.

- Conditions representing contraindications include also:
• Osteomyelitis
• Implantation in necrotic surgical sites
• Degenerative bone disease
• Intra-articular implantations
• Opening meninx
• As with any surgical procedure, care should be exercised in treating individuals with preexisting conditions that may affect the success of the surgical procedure.

Adverse effects

Possible adverse effects include but are not limited to:
• Wound complications including haematoma, infection, and other complications that are possible with any surgery.

Preparation

Before use, 4BONE BCH must be hydrated without excess with physiological saline to prevent osmotic damage. If desired, 4BONE BCH may be mixed with patient's whole blood following hydration with physiological saline.

Sterilisation and conservation

4BONE BCH granules are supplied sterile in vials or syringes.
4BONE BCH is sterilized using beta irradiation. This product should not be re-sterilized. The sterilization process of this product using radiations was validated during its conception.

Warning And Precautions

This product is to be handled or implanted by trained qualified persons having read this instructions for use. 4BONE BCH is intended only for bony voids or gaps that are not intrinsic to the stability of the bone structure.

Retract syringe plunger. To facilitate delivery, 4BONE BCH granules should be wetted. Draw physiological saline up through filter cap until liquid level is slightly above level of 4BONE BCH granules.

Expel excess liquid by depressing plunger slightly.

When wetted, the 4BONE BCH granules are easily expelled from the syringe. Deliver the granules directly in the surgical site with the syringe.

Option B: Deliver to the surgical site using conventional instruments such as curettes or directly with the syringe.

Key to codes used:
Do not use if package is damaged
Single use
Use by
Manufacture date (including sterilisation)
Consult instructions for use
Batch code
Sterilized using irradiation
Catalogue number
By prescription only
Manufacturer
Caution: Federal law in USA and Canada restricts this device to sale by or on the order of a physician.

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MIS Implants Technologies Ltd.